



WORKAIR ACTIVATION PROCEDURE


1. Connect WorkAir to the PC via USB (the WorkAir electronics button will flash orange and vibrate, indicating that the device has not yet been unlocked)
2. Enter in our website www.dairlab.com/en
3. At the bottom of the homepage, to the right, click on “WorkAir Activation”

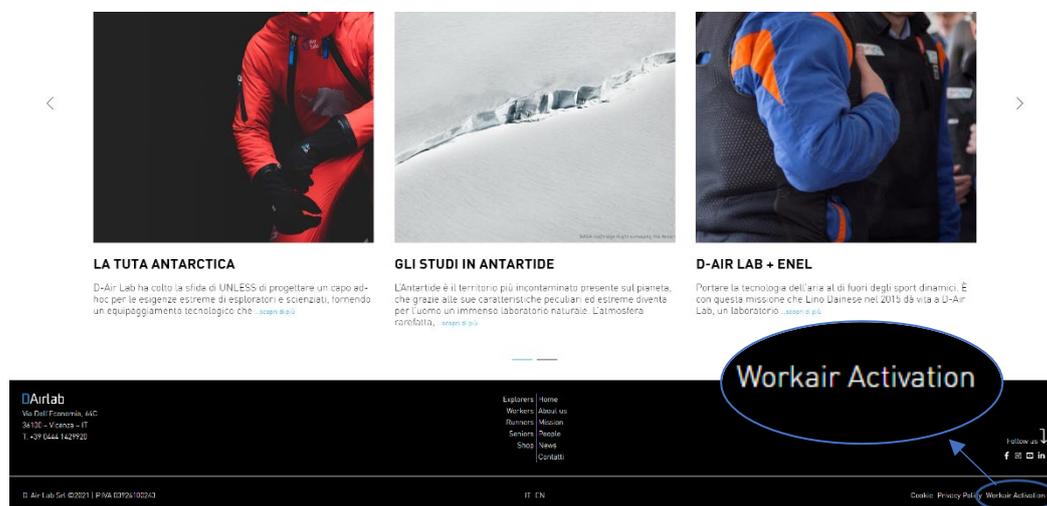


4. Click on “Connect WorkAir”
5. A pop-up will appear at the top of the page on the left: select the USB device (“*STM32 Virtual ComPort*”) and click on “Connect”

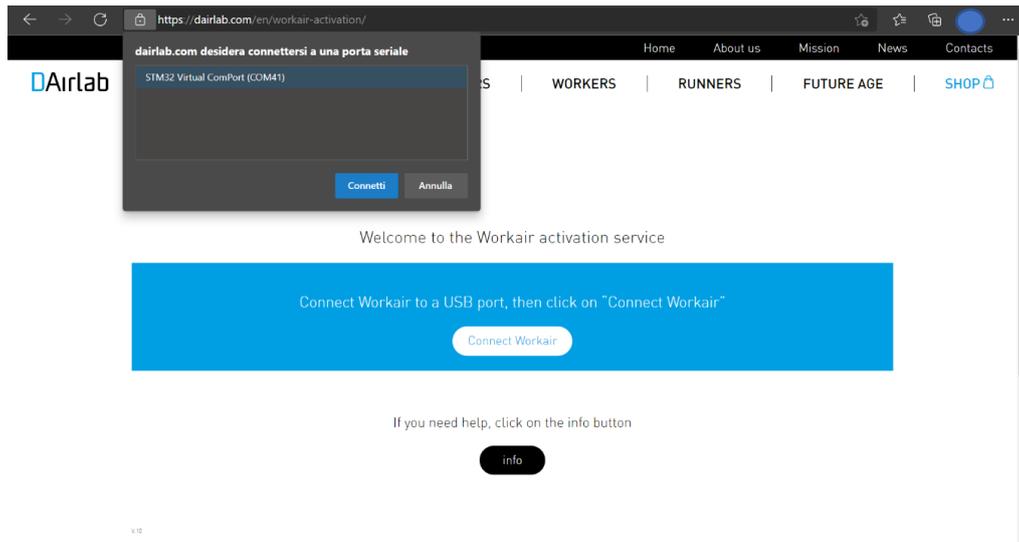

6. In the page it will be open will appear a green writing saying “WorkAir Connected” to your PC.
7. Click on “click here” to receive the Activation Code; if you already have the Activation Code, enter it in the appropriate box and click on “Activate”

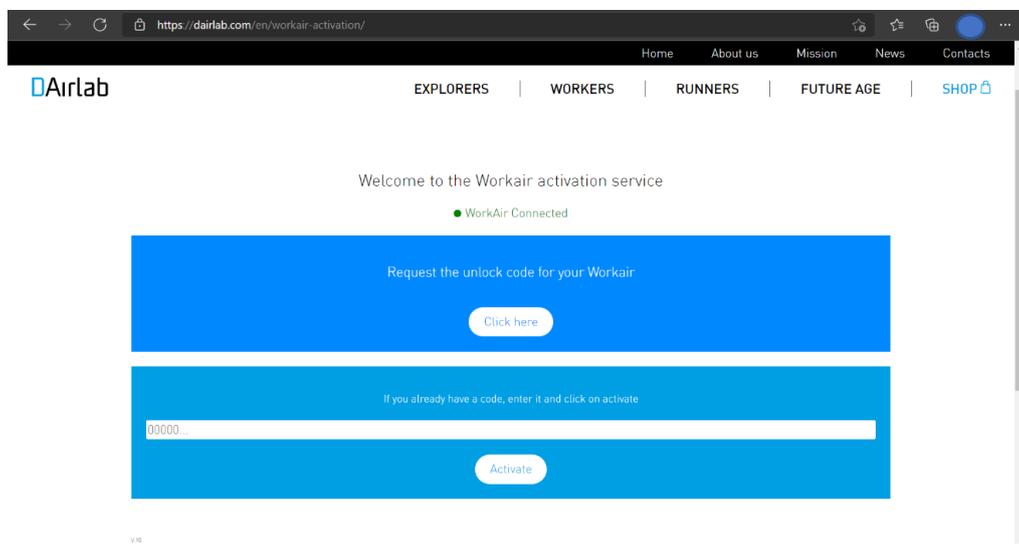


8. Manually insert your name, email and company
9. Click on “get serial number” and “get DairlabID”, the fields will be automatically completed

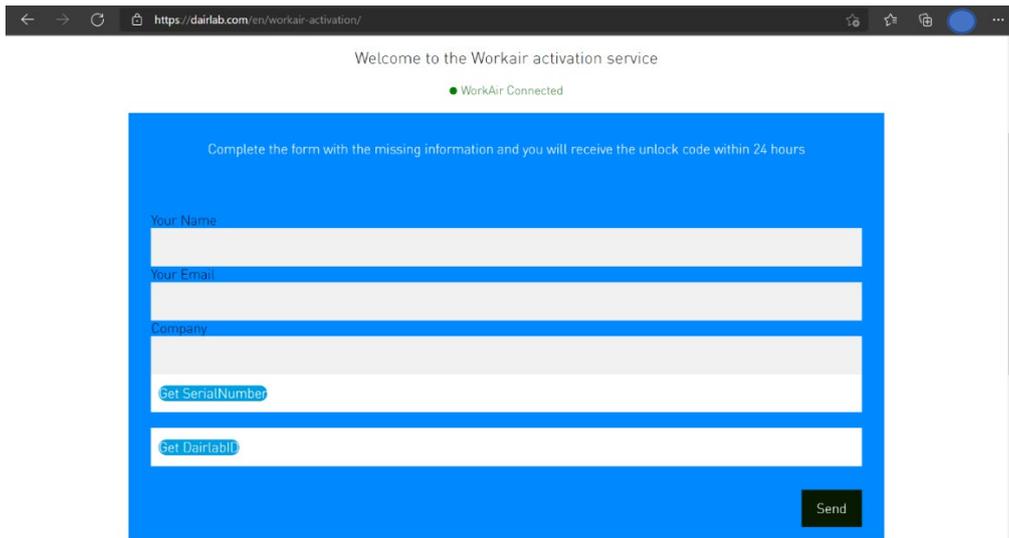

10. Click “Send”

11. In 24 hours you will receive an email with your Activation Code. You can copy and paste it in the apposite box, repeating the procedure from number 1 to number 7.

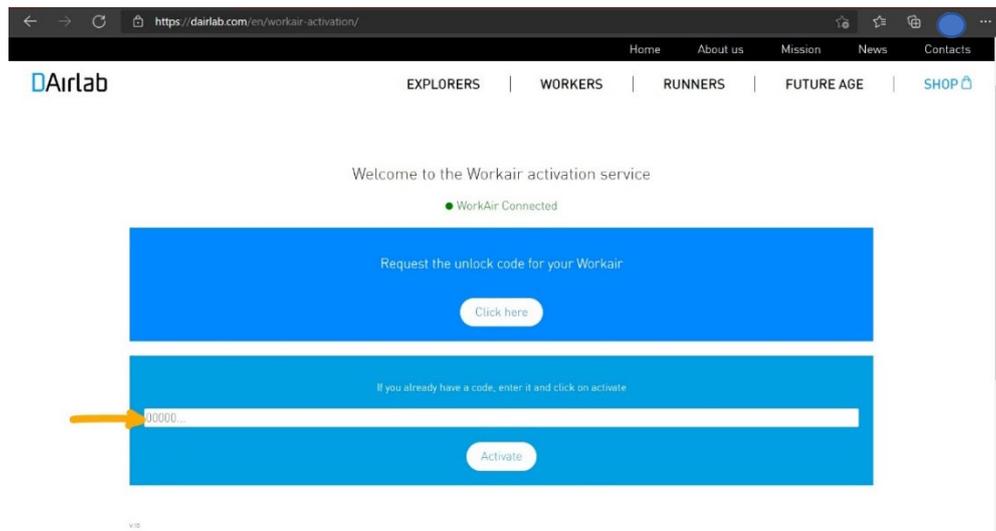

12. Click “Activate”

13. WorkAir is now activated! To ensure this: disconnect WorkAir from the PC, switch off the WorkAir using the button and switch it on again; if WorkAir first flashes blue and then green, orange or red (depending on the state of the battery charge), it is correctly activated and ready to use!

Good work with WorkAir!

Contact us if you have any difficulties with WorkAir's activation:
support@dairlab.com